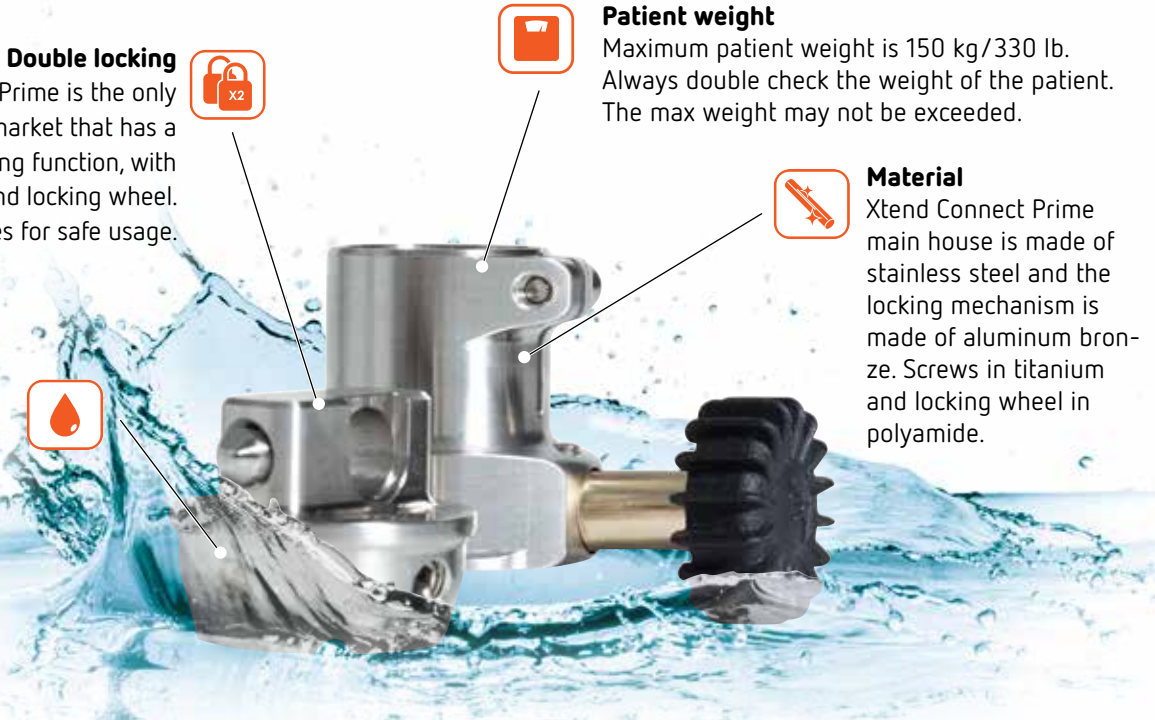


# Xtend<sup>®</sup> Connect Prime

## Technical Data Specification

Xtend Connect Prime is a patented quick adapter that allows the user to disassemble his/her prosthesis in parts to make it easier to change feet, change trousers or just save space, for example when traveling. Otherwise, Xtend Connect Prime works just like any adapter and has the common standard interfaces to fit most prosthetic products on the market.

Recommended L-code: L 5617










**Double locking**  
Xtend Connect Prime is the only product on the market that has a double security locking function, with both safety button and locking wheel. This guarantees for safe usage.

**Patient weight**  
Maximum patient weight is 150 kg/330 lb. Always double check the weight of the patient. The max weight may not be exceeded.

**Material**  
Xtend Connect Prime main house is made of stainless steel and the locking mechanism is made of aluminum bronze. Screws in titanium and locking wheel in polyamide.

**Water proof**  
All parts of Xtend Connect Prime are guaranteed for usage in salt-, pool and freshwater.

### Quick fit units and Base units

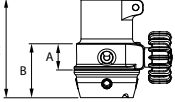
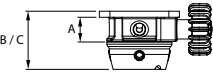
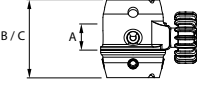
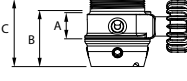
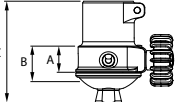
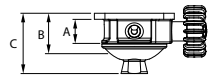
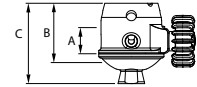
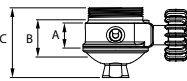
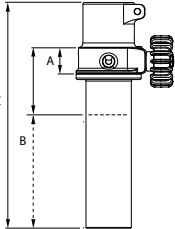
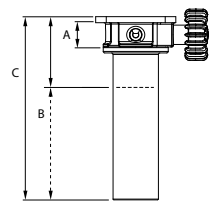
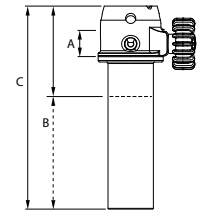
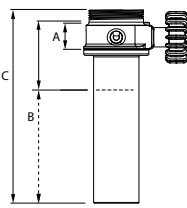
Item number	Description
 A100-Q30-150	Quick fit unit with tube clamp 30 mm, 150 kg/330 lb
 A100-Q4-150	Quick fit unit with 4-hole adapter, 150 kg/330 lb
 A100-QF-150	Quick fit unit with female pyramid adapter, 150 kg/330 lb
 A100-QP-150	Quick fit unit for prong adapter, 150 kg/330 lb
 A100-BF-150	Base unit with female pyramid adapter, 150 kg/330 lb
 A100-BM-150	Base unit with male pyramid adapter, 150 kg/330 lb
 A100-B30-150	Base unit with 30 mm tube, 150 kkg/330 lb

### Xtend Connect Prime Configurator



Not sure what components that are needed for a specific configuration? Scan the QR-code and use our online configurator.

# Technical facts – combinations

	<b>Q30</b> Quick fit unit, tube clamp 30 mm	<b>Q4</b> Quick fit unit, 4-holes adapter	<b>QF</b> Quick fit unit, female pyramid adapter	<b>QP</b> Quick fit unit, M36 adapter
<b>BF</b> Base unit, female pyramid adapter	 A:19 D:208 B:36 E:332 C:66	 A:19 D:208 B:39 E:346 C:39	 A:19 D:222 B:52 E:346 C:52	 A:19 D:208 B:37 E:309 C:45
<b>BM</b> Base unit, male pyramid adapter	 A:19 D:208 B:24 E:329 C:67	 A:19 D:208 B:27 E:344 C:40	 A:19 D:239 B:40 E:345 C:53	 A:19 D:208 B:25 E:306 C:46
<b>B30</b> Base unit, 30mm tube	 A:19 D:208 B:51-151* E:493* C:181	 A:19 D:208 B:54-154* E:507* C:154	 A:19 D:218 B:67-167* E:508* C:167	 A:19 D:208 B:60-160* E:471* C:160

**A** Functional height in mm **B** Effective height in mm **C** Total height in mm **D** Effective weight in g **E** Total weight in g  
\*B30: Adaptable effective height and weight as tube can be cut off with up to 100 mm.

## How to order

The Xtend Connect Prime is delivered with black locking wheel as a standard. For orange locking wheel add -O as a suffix to the item number. Maximum patient weight is 150 kg/330 lb. Always check that the patient's weight is correct. The maximum weight must never be exceeded. For intensive sports users, the maximum weight is 100 kg/220 lb.